

Instructions for use

Half mask with ELASTIC EAR BANDS.



Fig. 1



Fig. 2



Fig. 3



Fig. 4

Figure 1

Place the mask over the nose and over the mouth.

Figure 2

Position the elastic bands behind the ears, hook the rubber bands to the clip (if present) to improve comfort and prevent any leaks.

Figure 3

Fold the nose wire (or nosepiece) with your fingers to adapt it to the shape of the nose.

Figure 4

In order to test the mask's operation, put both hands on the mask then exhale vigorously. If air comes out from around the nose, press the nose wire (or nosepiece). If air escapes from the edges of the mask, reposition the clip (if present).

- Replace the mask immediately if breathing becomes difficult or if the mask is damaged or deformed.
- Replace the mask if you do not get an adequate seal on the face.
- Strictly complying with these instructions is an important step for the safe use of the mask.

Half mask with ELASTIC HEAD HARNESS.



Fig. 1



Fig. 2



Fig. 3



Fig. 4

Figure 1

Place the mask over the nose and over the mouth.

Figure 2

Pull the top strap over your head resting it high at the back of your head. Pull the bottom strap over your head and place it around your neck below your ears.

Figure 3

Fold the nose wire (or nosepiece) with your fingers to adapt it to the shape of the nose.

Figure 4

To check the seal between the mask and the face, wrap both hands over the mask and inhale sharply. In case of air leaks around the nose, tighten the nose wire (or nosepiece). In case of air leaks around the edges of the mask, slide the straps backwards along the sides of the head.

- Replace the mask immediately if breathing becomes difficult or if the mask is damaged or deformed.
- Replace the mask if you do not get an adequate seal on the face.
- Strictly complying with these instructions is an important step for the safe use of the mask.

Warnings

Before use, check the mask for any defects or imperfections; if in doubt, discard the mask and take a new one.

During use, avoid touching the mask directly: handle it using the elastic bands and in any case always after having carried out the hand hygiene procedure correctly.

Replace the mask with a new, dry and clean one as soon as the used one becomes damp. Remove the mask avoiding touching the front, using only the elastic bands.

Do not reuse used masks, dispose of them immediately after use.

Dispose of used masks in accordance with current waste disposal regulations. After disposal, carry out the hand hygiene procedure correctly.